

PIL-BOB[®]

Makes it easy to remove medication without spillage

Remove medication easily and hygienically without spillage


The Webstercare

"Innovative solutions to medication management"™

 webstercare

The Pil-Bob makes it even easier to remove pills hygienically and efficiently from

STEP 1

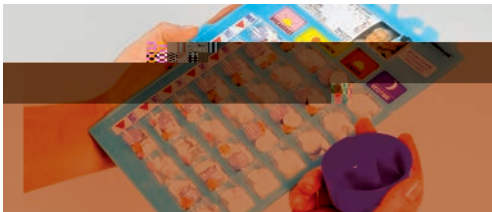


Hold the Webster-pak folder in a horizontal (flat) position

STEP 2



Place the Pil-Bob behind the blister you wish to open

STEP 3

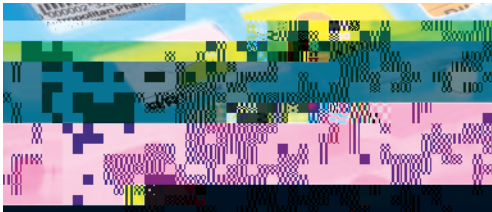

Insert the tip upwards through the foil backing

STEP 4



Sweep the blister with the serrated tip until all the pills drop out into the Pil-Bob cup